## ENGLISH – READ BEFORE USING THIS KIT: NEW ASSAY DEFINITION FILE (ADF)

- An updated ADF file has been released.
- Cartridges from this lot (and all subsequent lots) of Xpert® **Xpress** CoV-2/Flu/RSV **plus** require the import of ADF version 2 (called Xpress\_SARS-CoV-2\_Flu\_RSV\_plus\_2.zip) provided online at <https://www.cepheid.com/coronavirus-product-resources>. If you cannot download the ADF, you may order a CD with the ADF here <https://www.cepheid.com/en/contact/request-cd-international> or request a CD from Technical Support.
- All customers connected to a Laboratory Information System (LIS) may need to verify LIS outputs to ensure that they are consistent with instrument result outputs. Run quality assurance checks as determined by your institution's policy.
- The ADF update does not change the product intended use. Assay re-verification is based on your institution's requirements.

### For All Users: **IMPORTANT ADF Installation Instructions**

- The updated ADF can be run with both new and previous lots of product.
- When ready to import this ADF, please consult with your LIS Admin prior to installing ADFs.
- Take screenshots of the LIS Host Test Codes for each of the current ADFs (version 1). Version 1 ADF names include:
  - Xpress SARS-CoV-2\_Flu\_RSV\_plus\_1
  - Xpress SARS-CoV-2\_Flu\_plus\_1
  - Xpress SARS-CoV-2\_plus\_1
- Manually delete all 3 of the ADF files for the current ADF (version 1).
- Follow the ADF Import Instructions located on the product page to install the 3 ADFs for version 2.
- Enter the same LIS Host Test Codes as used for EUA ADFs (as shown on screenshots taken).
- If for some reason you believe you must revert to the older ADF, please contact Cepheid Technical Support for assistance:
  - Cepheid Technical Support (U.S.): + 1 888 838 3222
  - Cepheid Technical Support (Europe): + 33 563 82 53 19

### For US EUA Users:

- For use under Emergency Use Authorization (EUA) only.
- For prescription use only.
- For *in vitro* diagnostic use.
- This product has not been FDA cleared or approved but has been authorized for emergency use by FDA under an EUA for use by authorized laboratories.
- This product has been authorized only for the detection and differentiation of nucleic acids from SARS-CoV-2, influenza A, influenza B, and respiratory syncytial virus (RSV), not for any other viruses or pathogens.
- The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of *in vitro* diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb3(b)(1), unless the declaration is terminated, or authorization is revoked sooner.

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